



Dawson-Damer Park Water Play Space Wet Wheelchairs

Instruction document for wheelchair storage unit
and changing places amenities

Index

Wet Wheelchair use	Page 4
To position a person in the chair	Page 4
STEPS FOR SETUP	
Step 1 - Brakes	Page 5
Step 2 - Armrest	Page 5
Step 3 - Headrest	Page 6
Step 4 - Harness	Page 6
Step 5 - Release Brake	Page 7
After use	Page 7
Wet Wheelchair kinds	Page 7

Wet Wheelchair Use

The wheelchairs are approved for a maximum sitting user of 130kgs

Parents and carers of Wet Play users that are not toilet trained are required by NSW Health to:

- Wear aqua (swimming) nappies and/or tight fitting waterproof pants at all times;
- Ensure use of the toilet before entering the pool/aquatic facility;
- After changing, dispose of waste and wash your hands thoroughly with soap and sanitise.

To position a person in the chair, please follow these steps:


- All current COVID-19 restrictions must be adhered to at all times.
-
- Before any use of the wheelchair, check there is no red tag marked 'DO NOT USE' on the chair. If a red tag is present, do not use the wheelchair and report to Council immediately on 4654 7777.
- The wheelchairs are approved for a maximum sitting user of 130kgs.
- Before any use of the wheelchair, use antibacterial wipes to wipe down chair, check the wheels are correctly locked (by the clips), the straps are suitably tightened and the buckles are fastened.

STEPS FOR SETUP

Step 1

Brakes


- Ensure brakes are on by pulling out from under chair and clipping black handle under small bracket on frame.

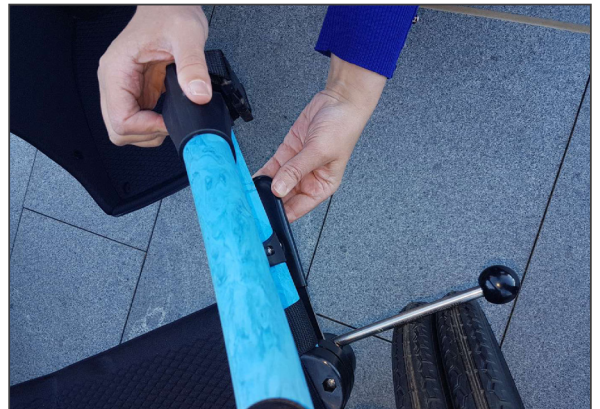

Step 2

Armrest

- To raise armrest, unclip strap and lift armrest straight up and to side.

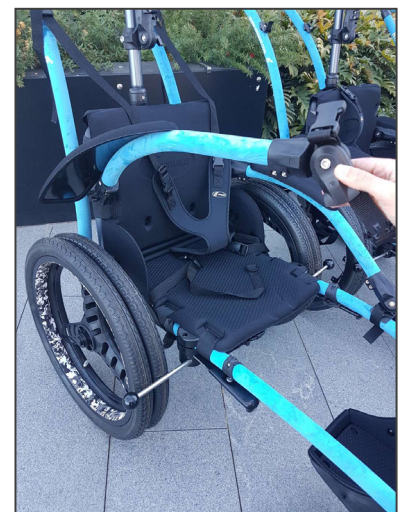

Step 3

Headrest

- Position person as far back as possible, with their back touching the backrest.
- Adjust position of headrest pad in contact with head and tighten all butterfly screws.

Use lower butterfly screw to adjust height and rotation, upper and central butterfly screw for forwards/backwards adjustment. Ensure headrest is stable.



Step 4

Harness

- Once person is seated correctly, adjust harness to fit firmly and clip it in to lock it.
- Check all straps and harnesses are suitably tightened and all buckles are fastened.



Step 5

Release brake

- To steer the chair, you can either skid front wheel (left or right) or by the assistant pushing down on handle to lift front wheel in order to change direction.



After Use


- Enter pin code to access storage facility.
- Return the chair to the storage facility.
- Use antibacterial wipes to wipe down chair.
- If there is damage to the chair, tie a red tag marked 'DO NOT USE' (available in the storage facility) to the damaged area and contact Council on 4654 4777 to report immediately.

Wet Wheelchair kinds


Hippocampe wet wheelchair -
pool chair



Hippocampe wet wheelchair -
medium chair

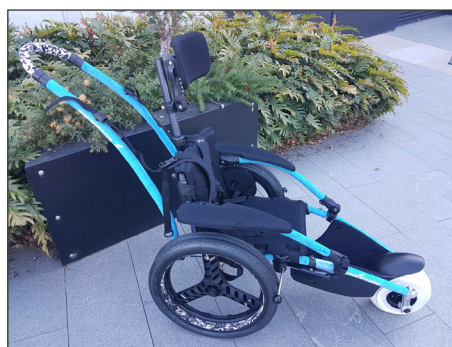

Hippocampe wet wheelchair -
large chair

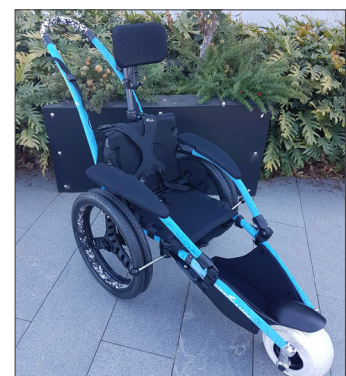



70 Central Ave,
Oran Park NSW 2570



PO Box 183,
Camden 2570



camden.nsw.gov.au



mail@camden.nsw.gov.au



4654 7777



www.facebook.com/camdencouncil